[EN] Consumer information

Stratacel is a flexible wound dressing for the repair of damaged or compromised skin following dermal abrasion, laser resurfacing and chemical peels.

Stratacel is designed for those areas of the skin that are especially sensitive. It is suitable for patients undergoing aesthetic interventions in the periorbital area, lips or nostrils or who have sensitive skin.

When used as directed Stratacel dries to form a protective layer that is gas permeable and waterproof which hydrates and protects compromised skin areas and superficial wounds from chemical and microbial invasion.

Stratacel can be applied to open wounds and areas where skin integrity has been compromised.

Stratacel is used to relieve itching and discomfort, and to reduce redness and discoloration associated with damaged skin.

How Stratacel works

Stratacel dries to form a protective layer which is gas permeable and waterproof.

Stratacel helps to promote a moist wound healing environment for compromised skin areas and superficial wounds. This environment leads to faster re-epithelialization and a reduced inflammatory response, which reduces the recovery time post-procedure.

Stratacel is a semi-occlusive, self-drying, transparent wound dressing which does not adhere strongly to the wound bed or the newly formed granulating tissue.

Stratacel gel is bacteriostatic and inert.

Intended use

Stratacel is a semi-occlusive wound dressing intended to cover a wound, to provide and support a moist wound environment, and to allow the exchange of gases such as oxygen and water vapor through the device.

Because Stratacel is a gel, it is easily applied to large exposed surface areas and contoured skin, like the face, neck and chest. Stratacel can be used with or without a secondary protective dressing.

Stratacel creates a moist wound environment.

Stratacel was developed to be used as a wound dressing for the repair of damaged or compromised skin following resurfacing procedures such as:

- Laser therapy (ablative and non-ablative), including tattoo removal
- Dermal abrasion
- Chemical peels

Stratacel may also be used in conjunction with other adjunctive treatments to improve overall results.

Stratacel contains no alcohols, parabens or fragrances, and is transparent and odorless.

Stratacel is suitable for people with sensitive skin. Stratacel may be used during pregnancy and while breastfeeding.

Directions for use

On first use of 10 g and 20 g tubes remove the cap, cut the nozzle tip of the tube and close with the cap after use. On 50 g tubes remove the cap, then the protective seal and close with the cap after use.

Ensure that the affected skin or superficial wound is clean.

Gently pat dry as much excess exudate or wound fluid from the area as possible prior to the gel application

Apply a very thin layer of Stratacel directly to the affected area and allow the gel to dry.

Stratacel should be applied once daily, or twice daily to exposed areas or as advised by your physician.

Once dry, Stratacel may be covered by sunscreen, cosmetics or clothing.

For best results Stratacel should be maintained in continuous contact with the skin (24 hours a day/7 days a week).

When applied correctly to exposed areas, Stratacel should be dry in 5–6 minutes. If it takes longer to dry you have probably applied too much. Gently remove the excess with a clean tissue or gauze and allow the drying process to continue.

Stratacel is intended for single patient use.

Recommended duration of treatment

Stratacel is recommended as immediate post-procedure treatment and should be continued until the skin has fully recovered.

For resurfacing and ablative procedures, continued use is recommended for a minimum treatment of 30 days (24 hours a day/7 days a week) or until no further improvement is seen.

How much Stratacel do I need?

Stratacel gel is a unique formulation which requires substantially less product per application than typical moisturizing creams or barrier ointments.

- **Stratacel 10 g (0.35 oz)** is enough to treat an area of 6 x 6 cm (2.5 x 2.5 inch) twice per day for over 60 days.
- **Stratacel 20 g (0.7 oz)** is enough to treat an area of 6 x 12 cm (2.5 x 5 inch) twice per day

for over 60 days.

- **Stratacel 50 g (1.75 oz)** is enough to treat an area of 12 x 15 cm (5 x 6 inch), twice per day for over 60 days.

Caution

- For external use only.
- Stratacel should not be placed in contact with the eyes.
- Stratacel should not be applied over topical medications unless advised by your physician.
- Stratacel may stain clothing if not completely dry. If staining occurs, dry cleaning should be able to remove it without damaging the fabric.
- For correct storage please reclose the tube tightly with the cap.
- Should your wound show signs of infection or fail to heal after 30 days of initial product application consult your physician.
- If irritation occurs, discontinue use and consult your physician.
- Not suitable for highly exudative wounds, tunnelling wounds or 3rd degree burns.
- Keep out of the reach of children.
- Do not use after the expiration (EXP) date printed on the tube. The expiration (EXP) date does not change once the tube has been opened.
- Do not use if the tube is damaged.

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Stratpharma AG